# all models

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Instructions

**Head mirrors and head lamps**

Mode d'emploi

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Art. Nr. 11302      Lampen 6 V für **clar N** LED

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Art. Nr. 11301      Pack. à 6 St. Lampen 6 V für **clar N** Vakuum

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Modelle:            **clar N** Vakuum 55 mm, **clar N** LED 55 mm  
                         **ri-focus®** LED

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**Caution: Observe the operating instructions**

**Device belongs to protective class II**

**Attention:  
Do not stare into the beam**

**Attention: Dispose of the device in accordance with valid regulations**

**Type B**

**Not for outdoor use**

**Alternating current**

**Direct current**

**LED light  
Do not stare into the beam  
Class 2 LED**

**LED Light  
Class 1 LED**

## 1. Important information to be observed before operation

You have purchased a high quality **Riester** product, which has been manufactured according to the Directive 93/42/EEC for medical devices and is subject to the strictest quality controls at all times. The outstanding quality is a guarantee for reliable diagnosis. Please read the Instructions for Use carefully before operating the unit and keep them in a safe place.

If you should have any questions, we or your responsible **Riester** product representative are available to answer queries at any time. Our address can be found on the last page of these Instructions for Use. The address of our sales representative will be provided upon request.

Please note that the instruments described in these Instructions for Use are only to be used by suitably trained personnel.

The user should ensure that the instrument is working safely and is in good condition before use.

Furthermore, please note that the perfect and safe functioning of our instruments is only guaranteed when both the instruments as well as the accessories used are from **Riester**.

### Remarks on electromagnetic tolerability

There are currently no indications that electromagnetic interactions with other devices can occur when this device is used correctly. However, interference cannot be completely excluded under the strong influence of unfavourable field strengths, e.g. when operating a cell telephone and from radiological instruments.

## 2. Headlights und head mirrors

### 2.1. Intended use

The headlights and head mirrors described in these Instructions for Use have been manufactured to illuminate or throw light on the areas of the body to be examined.

### 2.2. Headlights and head mirrors

#### Preparation prior to operation

#### Attention!

- Don't use the examination light for eye-examination
- There may be a risk of gas ignition when the light is used in the presence of inflammable drug mixtures such as air and/or oxygen or laughing gas!

### 2.3 Putting into initial service and function

#### 2.3.1 Adjusting the headband

**The adjustment of the headband is the same for all models.**

Loosen the aluminium turning knob in a counter-clockwise direction. Adjust the headband and fix it in the required position by turning the knob clockwise.

### 2.3.2 Adjusting the lamp head ri-focus® LED

Individual adjustment of the lamp head with the flexible light arm: see Picture 1 parallel to the optical path of the eyes.

Individual fine adjustment of the lamp head (during the examination) with the flexible light arm

#### **Caution!**

**The flexible arm of the light should not be bent in a too small radius in order to prevent premature material fatigue.**

**Don't continue with focussing when you realize the stop as this can damage the lamp.**

ri-focus® LED

### **clar N 55 mm**

Loosen the plastic turning knob on the joint, adjust the mirror and tighten the knob again.

clar N 55 mm

### 2.3.3 Fitting batteries or rechargeable batteries ri-focus® LED und clar N

In order to open the battery compartment, press the two push buttons and pull off the cover of the battery compartment

Polarity of batteries type AAA  
or rechargeable batteries Type AAA

Polarity of batteries lithium CR 123A

**Attention: It is important to ensure that the polarity is correct.**

Push the battery compartment cover back into the tracks of the battery compartment and push downwards until it snaps into place.

### 2.3.4 Switching on and off ri-focus® LED und clar N

Simple on-off switch located on the battery compartment.

### 2.3.5 Battery case cover

Battery case cover 1 is for  
batteries/accus type AAA (4 pieces)

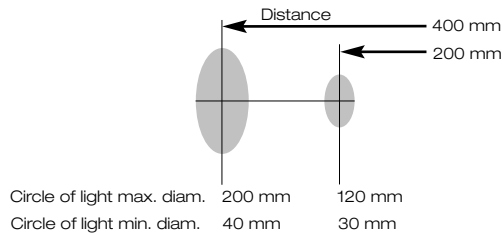
Battery case cover 2 is for  
batteries/accus type Lithium  
CR123A 2 pieces)

**Attention!** Please take care that there is no function if the wrong battery case cover is used.

### 2.3.6 Focussing ri-focus® LED

Focus by turning the front lamp head.

### clar N 55

Manually moving the lamp fixture towards and away from the mirror.

### 2.3.7 Attaching the headlight / the head mirror ri-focus® LED

The lamp head is firmly fixed to the headband and can only be exchanged together with the headband.

### clar N

By opening the plastic screw, the clamping unit located on the headband is opened up until the ball-and-socket joint on the mirror fits. Attach the mirror by turning the plastic screw tightly on the headband.

### Attention!

Before exchanging the complete mirror, the electric plug located on the joint of the lamp has to be pulled out. After the mirror has been exchanged, the plug can be pushed back in again.

If the lamp does not function after plugging the cable back in again, the plug has to be rotated 180° and plugged in again.

### Attention!

- Never touch the lamps during use.  
They can get very hot!
- In the case of model **ri-focus® LED** only touch the ring at the front of the lamp head or the grip at the rear of the lamp head during operation. All other parts can heat up considerably.
- In the case of **clar N** model, you can touch the following parts during operation: the plastic shell on the mirror, the adjusting knob and the swivel arm.

### 2.3.8 Lamps

#### Exchanging the lamp

##### clar N 55

Turn the lamp away from the mirror with the help of the adjustable lamp joint (enlarge the clearance). The lamp can then be unscrewed and a new lamp screwed back into place.

**Attention!** Allow the lamp to cool down first before exchanging the lamp!

### 2.3.9 Operation of the battery charger

#### ri-focus® LED and clar N 55

Plug the charger into the mains socket and then connect it to the battery compartment. As soon as a connection to the battery compartment has been made, the charging indicator of the unit lights up and the rechargeable batteries will be charged up.

**Charging time:** First charge minimum 24 hours.  
Following charges are possible over night or as required.

#### Attention:

- Only commercially available rechargeable batteries Type AAA may be charged with this charging device.
- Charging non-rechargeable batteries may lead to the destruction of the batteries or the charging device.
- The charging device may only be used in closed rooms.
- Remove the device from the mains point when not in use.
- Do not operate, if either the case or the mains plug is damaged.
- Do not open the device.
- If the device is not used for a longer period of time or is to be taken with when travelling, please remove the batteries or the rechargeable batteries from the battery compartment
- New batteries should be used or the rechargeable batteries should be charged up, if the light intensity of the instrument becomes weaker and could compromise examination results.
- In order to obtain an optimal light yield, we recommend the use of new, high quality batteries when exchanging batteries.

### 2.3.10 Technical data

Description	Voltage	Amperage	Average service life
Lamp for <b>clar N</b> vacuum 55 mm	6 V	0,4 A	approx. 200 hours
Lamp for <b>clar N</b> LED 55 mm	6 V	0,35 A	approx. 15.000 hours
LED for <b>ri-focus®</b> LED	6 V	0,525 A	approx. 50.000 hours

### 2.3.11 Exchanging the headband padding

The foam padding can be simply removed from the Velcro and replaced by new foam padding.

### 2.3.12 Instruction for care

#### General information

The goal of cleaning and disinfection of medical products is the protection of patients, users and third persons and conserving the value of the medical products. On account of the product design and the used material, no defined limit of maximum processing cycles can be fixed. The lifetime of the medical products depends on their function and on a appropriate treatment of the devices. Before returning faulty products for repair they must have gone through the described reprocessing process.

#### Cleaning and disinfection

Head mirrors and headlights can be cleaned outside with a humid cloth until optical cleanness is given.

Wipe disinfection according to the regulations given by the disinfectant manufacturer. Only disinfectants with approved efficiency and in accordance with the national standards can be used.

After disinfection the device must be cleaned with a humid cloth to remove all possible deposits of disinfectants.

#### Caution!

Never place the head mirror and headlights in liquid!

The items are not permitted for machine cleaning and sterilisation as they will be damaged!

## 3. Replacement parts

#### LED illumination

Art. No. 11302      Lamps 6 V for **clar N** LED

#### Standard illumination:

Art. No. 11301      Pack of 6 lamps à 6 V for **clar N** vacuum

## 4. Technical data

Models:            **clar N** vacuum 55 mm, **clar N** LED 55 mm  
                      **ri-focus®** LED

Energy sources:    See remarks on the respective mains device or on the batteries, rechargeable batteries

Initial values:     According to the details on the mains devices or the batteries used

Working temperature: +10 to +40°, relative air humidity between 30 and 75% (non-condensing)

Storage location:   -5°C to 50°C, up to 85% relative air humidity, non-condensing

## 5. Maintenance

The instruments and their accessories do not require any special maintenance. If an instrument has to be checked for

any reason, send it to us or to an authorised **Riester** dealer near you, whom we shall be pleased to name upon request.

## **6. Disposal**

Please note that batteries and electrical appliances have to be disposed of in a special way. You can obtain information about this from municipal collection points.

Manufacturer: see last page of these Instructions for Use.

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## **Warranty**

This product has been manufactured under the strictest quality standards and has undergone a thorough final quality check before leaving our factory.

We are therefore pleased to be able to provide a warranty of

### **2 years from the date of purchase**

on all defects, which can verifiably be shown to be due to material or manufacturing faults. A warranty claim does not apply in the case of improper handling.

All defective parts of the product will be replaced or repaired free of charge within the warranty period. This does not apply to wearing parts.

For R1 shock-proof, we grant an additional warranty of 5 years for the calibration, which is required by CE-certification.

A warranty claim can only be granted if this Warranty Card has been completed and stamped by the dealer and is enclosed with the product.

Please remember that all warranty claims have to be made during the warranty period.

We will, of course, be pleased to carry out checks or repairs after expiry of the warranty period at a charge. You are also welcome to request a provisional cost estimate from us free of charge.

In case of a warranty claim or repair, please return the RIESTER product with the completed Warranty Card to the following address:

**Rudolf Riester GmbH  
Dept. Repairs RR  
Bruckstr. 31  
72417 Jungingen Germany**

**Serial number or batch number:  
Date Stamp and signature of the specialist dealer:**

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