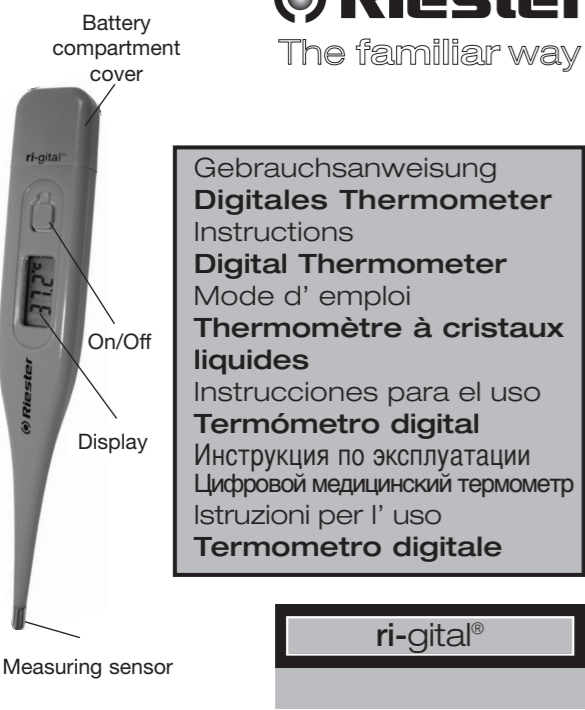
CE 0124

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#### 1. Important information to be read before using the thermometer

You have purchased a high quality Riester digital thermometer, which has been manufactured according to the directive 93/42 EEC for medical products and subjected to the strictest quality controls. Its outstanding quality will guarantee you reliable measurements for years to come

Please read these User Instructions carefully before first using the thermometer and keep them in a safe place.

We will be pleased to answer any questions you may have at any time. You will find our address on the warranty card in these User Instructions. We will be pleased to notify you of the address of our distributor on request.

#### 2. Important safety instructions

- The thermometer is suitable exclusively for measurement of the body temperature!
- The minimum measurement time until the signalling tone [beep] must be maintained without exception!
- The thermometer contains small parts (battery, etc.) that can be swallowed by children. Therefore never leave the thermometer unattended to children.
- Protect the thermometer from hitting and striking!
- Avoid ambient temperatures above 60°C. NEVER boil the thermometer!
- Warning:** The thermometer contains a long-life battery. For disposal, please comply with local regulations for special waste.
- The manufacturer recommends to verify the accuracy by an authorised laboratory every 2 years

#### 3. Monitoring of instruments

**EU countries except Germany:** Legal regulations for monitoring instruments apply to all EU countries except Germany.
**Countries outside the EU:** For any countries where no legal regulations exist for monitoring instruments, it is recommended to examine the accuracy of measuring instruments in 2-year intervals.

#### 4. Turning on the thermometer

To turn on the thermometer, push the button over the display window; a short signalling tone [beep] signals «thermometer ON.» A display test is performed. Then at an ambient temperature of less than 32°C, an «L» and a flashing «C» appear at the upper right of the display. The thermometer is now ready for use.

#### 5. Function test

Proper functioning of the thermometer is tested automatically each time it is turned on. If a malfunction is detected (measurement inaccuracy), this is indicated by «ERR»on the display, and a measurement becomes impossible. In this case, the thermometer must be replaced.

#### 6. Using the thermometer

During a measurement, the current temperature is displayed continuously. The «°C» symbol flashes during this period of time. If the signalling tone sounds (beep-beep-

beep-beep 10 times) and the «°C» is no longer flashing, this means that the measured increase in temperature is now less than 0.1 °C in 16 seconds and that the thermometer is ready for reading. To prolong the life of the battery, turn it off after use by pressing the operating button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

#### 7. Storage of Measured values

If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed. At the same time, a small «M» for memory will appear at the right of the display. After 3 more seconds, this value will disappear and the thermometer will switch to the normal measurement mode. Then the stored value is erased.

#### 8. Measuring methods

**In the mouth (oral)**

Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor must be in good contact with the tissue. Close your mouth and breath evenly through the nose to prevent the measurement from being influenced by inhaled/ exhaled air. Approx. measuring time: 1 minute.

**In the anus (rectal)**This is the most reliable measuring method, and is especially appropriate for infants and small children. Carefully insert the tip of the thermometer 2 to 3 cm into the anal aperture. Approx. measuring time: 1 minute!

**In the armpit (axillary)**

From a medical viewpoint, this method will always provide inaccurate readings, and should not be used if precise measurements are required. Approx. measuring time: 5 to 10 minutes!

**CLEANING AND DISINFECTING**

The thermometer has a waterproof tip. For disinfecting please wipe with a alcohol moist towel.

#### 9. Battery Replacement

When the «▼» symbol (upside-down triangle) appears in the lower right of the display, the battery is exhausted and needs replacing. For the battery replacement please remove the compartment lid of the thermometer. Replace the exhausted battery by positioning the + at the top. Make sure a battery of the same type is on hand. Batteries can be purchased at any electrical goods shops. For battery disposal please refer to «safety instructions».

#### 10. Technical specifications

Type: Maximum thermometer
Measurement range: 32.0°C to 43.9°C
Temp.< 32.0°C:display «L» for low (too low)
Temp.> 43.9°C:display «H» for high (too high)
Measurement accuracy: ± 0.1°C between 34°C and 42°C at an ambient temperature of 18°C to 28°C

Self-test: Automatic internal check at a test value of 37.0°C; if there is a deviation of >0.1°C, «ERR» (error) is displayed.

Display: Liquid crystal display (LCD) with three digits Smallest unit of display: 0.1°C

Signalling tone: For signalling that the thermometer is ready to use and that the temperature increase is less than 0.1°C/16 seconds.

Memory: For storing the last measured value.

Storage temperature: –10°C to +60°C

Battery: 1.5/1.55 V

Reference to standards: EN12470-3: 2000, clinical thermometers ASTM standard

 Meaning of the symbol at the back of the case: **Please pay attention to the user manual**

 Meaning of the symbol at the back of the case: **Applied part Type B.**

 Used electrical and electronic products are not to be disposed as unsorted municipal waste and are to be collected separately accordingly to national/EU regulations.

**EMC-Information**

The thermometer complies with the EMC requirements according IEC 60601-1-2. Portable and mobile high frequency appliances can influence medical electronic devices. Medical-electronic devices need special safety requirements concerning EMC. You can get a detailed description of the EMC-requirements at [www.riester.de](http://www.riester.de).

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<b>Rudolf Riester GmbH</b> <b>Reparaturen Abt. RR</b> <b>Bruckstr. 31</b> <b>D-72417 Jungingen</b> <b>Deutschland</b>
<b>Seriennummer bzw. Chargennummer</b>
<b>Datum</b>
<b>Stempel und Unterschrift des Fachhändlers</b>
<b>Warranty</b>
This product has been manufactured under the strictest quality standards and has undergone a thorough final quality check before leaving our factory. We are therefore pleased to be able to provide a warranty of
<b>2 years from the date of purchase</b>
on all defects, which can verifiably be shown to be due to material or manufacturing faults. A warranty claim does not apply in the case of improper handling.
All defective parts of the product will be replaced or repaired free of charge within the warranty period. This does not apply to wearing parts.
A warranty claim can only be granted if this Warranty Card has been completed and stamped by the dealer and is enclosed with the product.
Please remember that all warranty claims have to be made during the warranty period.
We will, of course, be pleased to carry out checks or repairs after expiry of the warranty period at a charge. You are also welcome to request a provisional cost estimate from us free of charge.
In case of a warranty claim or repair, please return the RIESTER product with the completed Warranty Card to the following address:

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<b>Seriennummer bzw. Chargennummer</b>
<b>Datum</b>
<b>Stempel und signature of the specialist dealer</b>
<b>Warranty</b>
This product has been manufactured under the strictest quality standards and has undergone a thorough final quality check before leaving our factory. We are therefore pleased to be able to provide a warranty of
<b>2 years from the date of purchase</b>
on all defects, which can verifiably be shown to be due to material or manufacturing faults. A warranty claim does not apply in the case of improper handling.
All defective parts of the product will be replaced or repaired free of charge within the warranty period. This does not apply to wearing parts.
A warranty claim can only be granted if this Warranty Card has been completed and stamped by the dealer and is enclosed with the product.
Please remember that all warranty claims have to be made during the warranty period.
We will, of course, be pleased to carry out checks or repairs after expiry of the warranty period at a charge. You are also welcome to request a provisional cost estimate from us free of charge.
In case of a warranty claim or repair, please return the RIESTER product with the completed Warranty Card to the following address:

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